

CollectEject™ Oral Sample Collection Instructions

Package Contents:

- i. CollectEject Swab sealed in wrapper, sterile
- ii. GenDry™ CE Pouch with a desiccant inside

1



Do not eat, drink, or smoke for 30 minutes prior to collecting the sample

2



Peel open the package containing the swab and stop peeling where indicated

3



Remove the swab. Be careful not to press the green plunger or touch the swab head. Do NOT discard the package

4



Place the swab head under your tongue for 20 seconds



Rub the swab head against your cheek for 20 seconds



Rub against the other cheek for 20 seconds

5



Reinsert the CollectEject Oral Swab headfirst into the original package saved from step 3.

Make sure the swab head is secured in the non-peeled part of the package

6



Open the GenDry Pouch at the tear-notch and unzip.

Fold over the top of the opened package to easily insert into the GenDry Pouch containing the desiccant (do not remove) and reseal

7



Let the sample stabilize for 24 hours. Process the sample with your protocol



[Video instructions:](#)
[Scan QR code to open](#)

Support
Ph: +1.608.949.8568
Email: Support@gentueri.com